
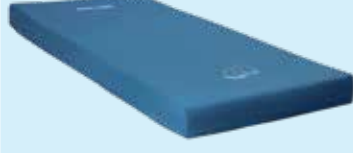
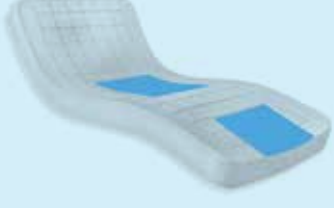
ARIANAMED

Mattress

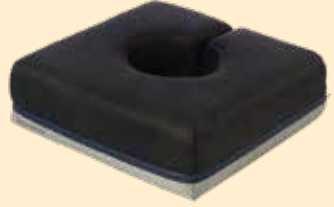
Cushions

Restraint Systems

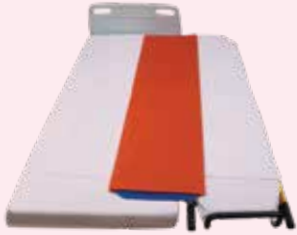Transfer Roll

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Introduction

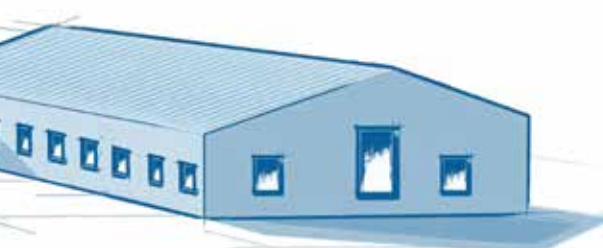
Ariana Tashkhis Abzar was established in 2010. Focused all its efforts on sore prevention and patient care products. The company expands product lines and improving upon existing memory polymers and chemical formulate to deliver superior sore prevention products. Ariana also has a wide range of restraint systems which have been used in different hospitals and health centers.

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Sectors: Hospitals, Clinics, Health Centers, Emergency, Orthopedics, Home Care, Elderlies

More info:

www.arianamed.com
www.aparat.com/arianamed

www.instagram.com/tashkhisabzar
www.digikala.com/search/arianamed
[www.linkedin.com/ariana tashkhis abzar](https://www.linkedin.com/ariana-tashkhis-abzar)

ARIA-PRO+ Memory Foam Mattress:

Two-layer mattress. Visco-elastic thermal foam which is located between a 3D spacer fabric and a cold foam core. Visco-elastic layer reacts to pressure and warmth and adapts to the body in a optimize shape. The mattress adapts to the contours of the body for optimum distribution of pressure and reacts to the patient's every move. This leads to a better blood circulation and reduces the risk of decubitus.

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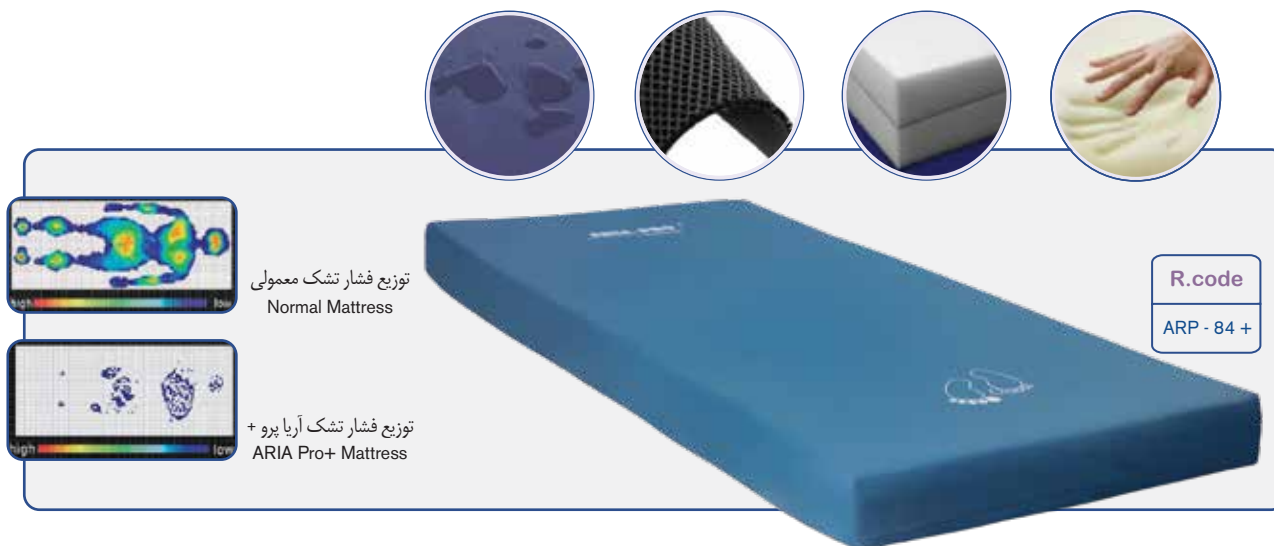
Technical Specifications:

- Suitable for high risk patient (up to stage 3)
- Viscoelastic foam with 85 kg/m^3 density
- High Resilience Foam with 50 kg/m^3 density
- Use 3D fabric to reduce moisture
- For patients from 30 to 200 kg (66 to 440 lbs)
- Non Energetic
- Two years guarantee for foam and a year for cover
- Dimension $200 \times 90 \times 16 \text{ cm}$ (more size upon request)

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Technical Specifications:

- Suitable for medium to high risk patient (up to stage 2 - 3)
- Viscoelastic foam with 85 kg/m³ density
- High Resilience Foam with 50 kg/m³ density
- Use 3D fabric to reduce moisture
- For patients from 30 to 180 kg (66 to 396 lbs)
- Non Energetic
- Two years guarantee for foam and a year for cover
- Dimension 200*90*14 cm (more size upon request)

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ARIA-HR Mattress:

ARIA hospital mattress is made of flexible and high density High Resilience foam with 50 kg/m³ density. This mattress reduces the pressure and minimizes the risk of developing ulcer and it is suitable for Low Risk patients. It has maximum comfort in all position for patients.

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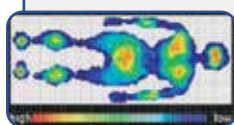
Technical Specifications:

- Suitable for Low Risk Patients (up to stage 1)
- High Resilience Foam with 50 kg/m³ density
- For patients from 30 to 130 Kg (66 to 288 lbs)
- High flexibility
- Two years guarantee for foam and a year for cover
- Dimension 200*90*10 cm (More sizes upon request)

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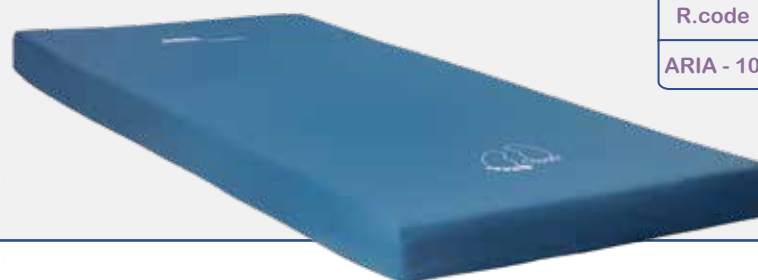
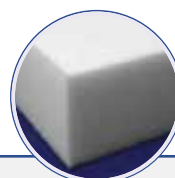
ARIA - 10

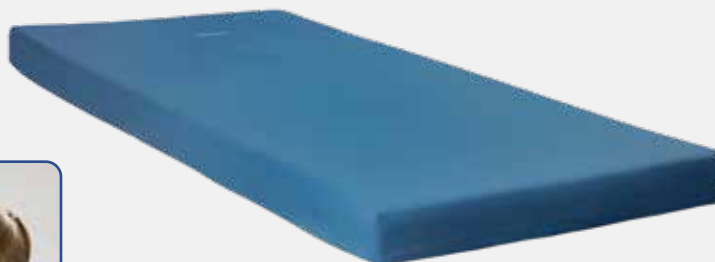
ARIA-Ped Memory Foam Mattress:

- Anti allergy for pediatrics skin
- Memory foam reducing pressure
- Suitable for PICU and pediatric wards
- Two years guarantee for foam and a year for cover
- Dimension 165*70*14 cm (more sizes upon request)

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ARIA - Ped

ARIA-Neo Specifications:

- Memory Foam in 4 cm with ability to reduce infant's pressure
- Anti Allergy for neonate skin
- Can be used in incubator, resuscitation and infant bed
- Two years guarantee for foam and a year for cover
- More sizes upon request

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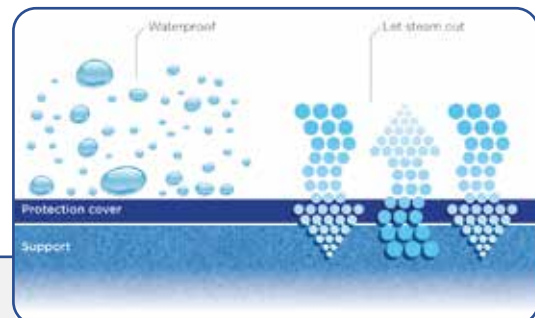
- Flame Retardent
- Antibacterial and Anti Fungi
- Anti-Allergy
- Anti-Static

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Medical Cover:

The cover accord with high medical quality is designed for hospitals/clinics and home remedies. Medical cover is used for alternative pressure mattress too.

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Model	See Page	Hight	Max Patient Body Weight	Expected Decubitus Risk			
				Low	Medium	Medium / High	High
Aria Pro+	2	16	200				●
Aria Pro	3	14	180			●	
Aria	5	10	130	●			
Aria Ped	6	14				●	
Aria Neo	6	4	8				

Aria Cube Series Mattress Features:

- Ease of installation and application
- High patient comfort and ability to move or replace gelpad or memory based on patient pressure ulcer risk
- Permanent air flow under patient body for reduction of surface pressure
- Foam surface non connectivity allows each section to have higher flexibility while in contact with patient body.
- The pads used result in lowering pressure unto very high risk pressure ulcer
- Long shelf life of gelpad and foam
- No need for repair or connection to electrical source
- Two year guarantee for replacement of foam and pad

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
Risk Table of Aria Cube Series Mattress:

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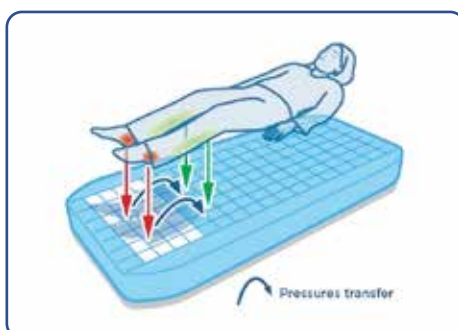Aria Cube Cold Foam Mattress:

Aria Cube cold foam mattress with Gelpad is suitable for high risk pressure ulcer and with its replacement with memory foam it is proper for medium/ high risk. If only one cold foam layer is used then the mattress is suitable for low risk.

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Ariana Memory Foam Pillow:

Ariana pillows consist of high density memory foam which conform to fit your unique profile, resulting in cradling effect that holds your head and neck in neutral alignment. Ariana pillows reduce the Neck pain and allow you to change positions without losing out on support as well.

- It is available in Large and XLarge size

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Memory Foam Heel Support Boot:

Ariana memory foam heel support boot is used to cases in which the lower organs are immobilised, as part of sore prevention. These boots are also used to prevent foot drop which is very common problem in ICU'S and CCU'S.

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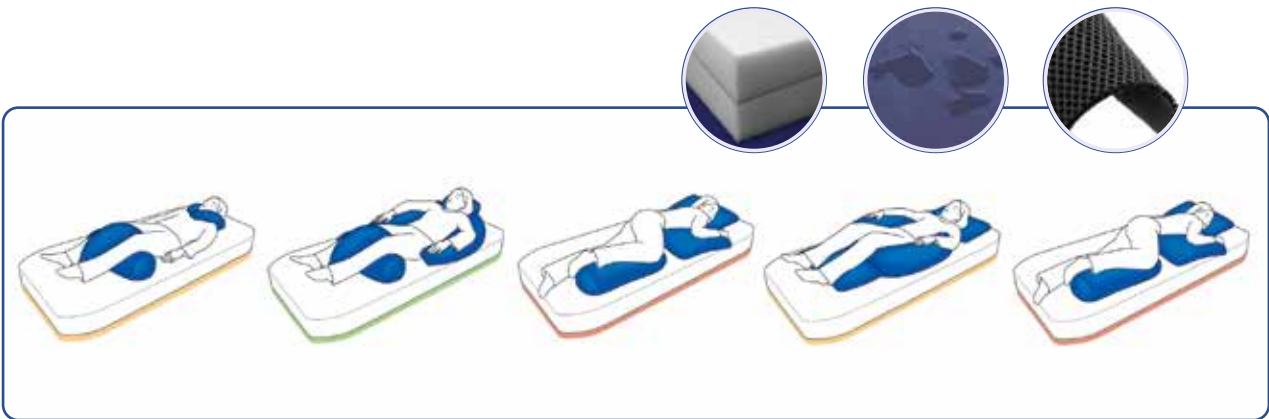
Intensive care positioning cushions consist of tow different layers, includes HR and memory foam. Besides positioning different patients these cushions also reduce the pressure of different parts of body and therefore reduce the risk of bedsores. Spacer layer also reduce the sweat and PU cover is washable and bactrical retardant.

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Large & Small Cushion:

Small and Large Cushions. This form of cushion is the perfect supporting aid. It helps support the patient, relieves the pressure on the heels, protects the internal lateral knee areas, ankles and can be used as pillow used by itself or with other devices.

R.code	Size
AR 902	Large
AR 903	Small

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Cylindrical Cushion:

It can be used in various ways, such as placing it under the knees to maintain lower limbs in semi-fowler positions.

R.code	Size
AR 904	Large
AR 907	Small

Neck Holder:

Comfortably supports the head both in sitting or semi-reclined positions.
It is available in Large and Xlarge.

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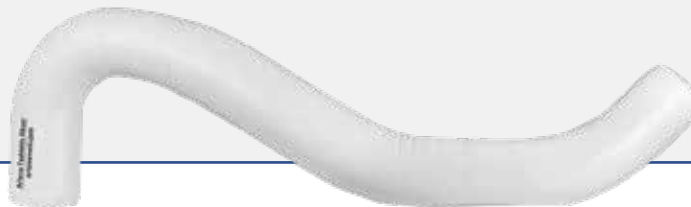
Lateral Positioning Cushion:

- Design with integrated pillow for optimal comfort.
- Ensures comfortable positioning of the knees, pelvis and upper body when in lateral position.
- Also it can be used for women during pregnancy.

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R.code
AR 906

Foam Chest Roll:

Two-Layer Chest Rolls Consist of HR and memory foam, reduce the pressure and help the operator to position the patients in different wards.

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R.code
AR 047
AR 043
AR 046

ARIANA Memory Foam Lumbar Cushion:

Arianamed support cushion uses an adjustable strap and soft high quality memory foam. This means better support and ultimate comfort and less pain. This cushion reduces stress on lower spine from improper posture. Relax your back and relieves lower back. This cushion has adjustable strap for different wheelchairs and chairs. Size: 46*38*4 cm

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Wheelchair Cushion:

The memory foam wheelchair cushion is used to avoid bedsores in patient's body who is confined in a wheelchair for a long time.

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All cushions are available with PU cover for hospitals and health centers.

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ARIANA Secure Fasten Wrist Holder With Ring:

Prevents pressure being exerted on the wrist. With safety D-ring. Inner layer made of %75 polyester and %25 viscose to avoid allergies. Breathable and comfortable. Easy to adjust. Great restrain properties. Can be used interchangeably to restraint wrists or ankles.

• Benefits

Better tolerated by the patient. Prevent skin injuries by laceration and abrasion. Made of Hypoallergenic material. Saves time to nurses. More safety and comfort for the patient.

R.code	Size
AW 3010	Adult
AW 3011	Infant

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ARIANA Abdominal Safety Belt:

It is flexible and adapts to patient's waistline without bending. Safe and easy fastening by means of very strong polyester straps and light nylon rings. Fully padded with soft and breathable tissues. Inner layer made of %75 polyester and %25 viscose. Capable to absorb pressure maintaining its performances.

• Benefits

It does not strangle the patient and allows the postural movements. Facilitates nursing work. Comfortable for the patient.

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ARIANA Anklebands:

Restraining neoprene ankle bands with double Velcro closure. Prevent injuries and falling out of bed or stretcher due to psychiatric restraint problems. Closure with safety clip on the ankles.

ARIANA Wristbands:

Restraining neoprene wrist bands with double Velcro closure. Prevent injuries and falling out of bed or stretcher. Closure with safety clip on the wrists. Since psychiatric restraint is a delicate matter.

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ARIANA Wristbands:

The Ariana wristbands allow to ensure the patient's hands to the bed. These are fixed to the bed structure by a long adjustable band.

- The Controlled Mobility System work combined with the abdominal belt, passing the wristband band through the lower belt loop, thus limiting the movements outside the perimeter of the bed.
- The inside of the wristbands is made of a soft touch padding, preventing pressure sores or abrasions.
- The wristbands contains an area with Velcro® that allows faster and tight application, increasing system security.

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ARIANA Anklebands:

The Ariana anklebands allow to ensure the patient's feet to the bed. The set composes a strap fixed to the bed structure and two independent ankle bands that are fixed to the bed by locks.

Wristbands			Anklebands		
R.code	Color	Size	R.code	Color	Size
3050	●	S	3060	●	S
3051	●	M	3061	●	M
3052	●	L	3062	●	L
3053	●	XL	3063	●	XL

4 Lock buttons

1 Magnetic key

Harness with Thorax Band:

It inhibits patient vertically upward movement and is designed to be very flexible.

Thorax band hinders slipping and harness to be loosen.

The transversal strap part of the restraint stops fast and violent patient movement. Therefore, it prevents patient self injury or injuring others.

If patient is non violent the transversal strap can be removed.

This allows the patient to sit up and prevents slipping up.

If fixation to the belt is not used, harness bands can be used in parallel position. That is they are used in an uncrossed position.

This allows the restraint not to touch large patient's neck or prevent pressure on the patient's sternum.

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AWD3080

ARIANA Complete Set:

The complete set is the global solution of secure restraint for bedridden patients. It permits having the maximum level of security of the patient and the medical staff, because of bed restraint is a not easy issue. As consists of individual elements can be used as required at the time.

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Complete Set for Ambulation:

Restraint consists of ankle bands, binding band, and a set of wristbands sewn onto the belt band. This restraint is designed to prevent patient aggression, manipulation, self injury, or access to the locks. It also holds the patient's leg and arms in a convenient position. It enables the patient to sit as well as taking short steps.

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ARIANA Wheelchair Restraint Belt:

The belt have been designed to provide a simple but firm subjection to weak or unstable patients. ARIANA belts have different models to adequate to different needs. It is adjusted to the chair by means of strong polyester straps on both sides and tied together at the back of the seat. The belt is washable up to 40°C.

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ARIANA Patient Restraint Belt:

For Bed or Strecher :

Articulated belt which secures the patient from falling down from bed or strecher allowing the patient to turn or change position. It is adpated to the patients waist by Velcro and an adjustable clasp.

- It is available in Large and XLarge strip

R.code	Size
ABD5000	XL
ABD5001	L

Transfer System Ariana:

Ariana Abdominal restraint belt is designed in two models of one and two parts and is used for prevention of patient from falling from the surgical bed or patient transfer beds.

The restraint belt is made of Neoprene material which reduces the pressure to the abdomen and as a result provides comfort for the patient.

This restraint belt is proper for use in surgical recovery rooms, on patient transfer beds, and on ICU beds.

In addition for use of this product on abdomen, it can be used to harness knee and leg

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Transfer System Ariana:

Innovative device that provide quick restraint with secure and safe transfer of altered, violent, aggressive or disoriented patients on emergency stretchers. It allows the transfer of patient from the emergency stretcher to other stretcher or bed, without release the restraint.

It consists in a resistant and durable mat provided of a set of harness, wristbands and ankle bands, with quick double velcro closure system and sliding base to improve the transfer.

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AWD3010

ARIANA Patient Transfer System:

The board with sliding sleeve for lateral transfers Ariana-Roller can comfortably transfer patients between beds, stretchers, operating tables, X-ray tables, etc. Patients may transfer up to 135 kg between two surfaces with a gap of 20 cm and spaced 20 cm. With improved core and replaceable outer sheath with very low friction and easy sanitation.

- New Core
- X-ray Transparent
- Less Weight
- More Comfortable
- Size:
- Full Board: 180*50 cm
- Half Board: 80*40 cm

R.code	Size
ARM 4011	40*80
ARM 4010	50*180

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Step 2

Step 3

One Time Installation of Antibacterial Curtain:

This curtain is covered with Ag (Silver) Ion that results in bacteria removal and reduction of transmission probability of pathogenic disease factors from the curtain surface to the patient or ward personnel and from other surfaces to the curtain.

The textile used in simple curtains in hospitals usually have the potential to become a center for aggregation of different bacteria, for example they can stay on these curtains for a long time.

Manufacturing is based on the standards of antibacterial textile AATCC100 and AATCC147 and it is effective on different micro organisms in both simple and antibacterial models:

- Simple models is used in hospital wards with low infection risk with large patient volume and Antibacterial model is used is suitable for special care and infectious (isolation) wards.
- Ease of installation and compatibility with hospital wards
- Provided in different colors: Green, Blue, Yellow ●●●
- Available in three different sizes (small 2.5, medium 5, large 7.5) with 2 meters height

*Arianamed suggests replacement of the curtain every 6 to 12 months. This is dependent on different wards' work load and patient volume.

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